

INSTRUCTIONS FOR USE

CONDAAF ACID

37% Phosphoric Acid Conditioner Gel

1. PRODUCT DESCRIPTION

CONDAAF Acid 37% AF do BRASIL is a low viscosity phosphoric acid conditioning gel, which does not drip from the applied area, and also has thixotropic properties. It has blue dye to facilitate visualization and control during application. Hydrophilic product as it has a good affinity with water, thus allowing easy removal after conditioning. It does not flow from the applied area.

2. PRESENTATION

1 syringe containing 3g of CONDAAF ACID.

3. COMPOSITION

Phosphoric Acid, Polyethylene Glycol, Deionized Water and Methylene Blue Dye.

4. INDICATION OF USE

Indicated for acid etching of enamel and dentin, providing increased adhesion and retention of adhesive systems on the surface.

5. PRODUCT ACTION

ACIDO CONDAAF is a gel containing Phosphoric Acid at a concentration of 37%, which reacts with the enamel or dentin surface forming soluble Calcium Phosphate salts.

These soluble Phosphates are removed and thus the surface is free of loose or adsorbed particles, presenting itself as a rougher relief and also more porous (especially in the case of dentin, where the dentinal tubules are unobstructed and exposed).

Cleaning associated with roughness and porosity conditions are the factors that improve the performance of adhesives on the conditioned surface. With the surface clean and well exposed, the adhesive penetrates more easily.

6. CONSERVATION/ STORAGE

Keep the product in its packaging always closed. Protect from direct sunlight.



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

7. INSTRUCTIONS FOR USE

Promote prophylaxis, wash and dry the area to be conditioned well.

Apply the CONDAAF Acid in the region to be etched and let it act for 30 seconds on the enamel and 15 seconds on the dentin.

Fluoridated teeth may require a second application.

After conditioning, wash the surface with plenty of water and then dry with gentle oil-free air jets.

8. CONTRAINDICATIONS

Contraindicated for people with reported sensitivity to any of the product's components.

9. WARNINGS

- Do not use after expiration date;
- Keep out of the reach of children;
- Exclusive use of the dentist.

10. SPECIAL CARES

Avoid contact with skin, oral tissues and eyes, if this occurs wash immediately with running water.

11. MANIPULATION

- When handling the product, the professional must wear protective gloves and goggles;
- Dispose of the product in accordance with current legislation in the country.

12. INTERACTIONS

There is not.