

INSTRUCTIONS FOR USE

LIGHT-CURING GINGIVAL BARRIER

1. PRODUCT DESCRIPTION

Gingival Barrier AF do BRASIL is a light-curing resin, it does not cause irritation to the patient's gingival tissues. It has adequate viscosity and thixotropy to allow easy and perfect coverage of soft tissues, without running to undesirable regions. Excellent sealing power. Minimal and perfect adhesion on the gum. Does not release from gum unless forced. It comes out completely when you take it off. Yields up to 5 full-arch applications.

2. PRESENTATION

In black polypropylene syringe containing 2g, 2.5g, 3g, 4g or 5g. In single packs or containing 2 units; 3 units; 5 pcs; or 10 pcs.

3. COMPOSITION

Uretame Dimethacrylate Monomers, Methacrylate Monomers, Silica, Inert Load, Pigments, Dyes and Photoinitiators.

4. INDICATION OF USE

Indicated as a substitute for the rubber sheet for the protection of the gingival tissue and in cases of risk of irritation due to contact with products used in the clinical procedure, such as, for example, tooth whitening and as an adjunct in absolute isolation for better sealing of a cavity preparation or endodontic treatment.

5. PRODUCT ACTION

The main purpose of the product is to protect soft tissues from contact with hydrogen peroxide (whitening gel).

6. CONSERVATION/STORAGE

- Keep the product in its packaging always closed;
- Protect from direct sunlight;



STORE IN A DRY PLACE AT ROOM TEMPERATURE(15 TO 30°C).

7. INSTRUCTIONS FOR USE

- Dry the gums and teeth corresponding to the region you want to protect;
- Apply a thin layer of the product (1-1.5 mm thick) on the gingiva and gingival papillae in such a way as to obtain total coverage and a good field of protection for the procedure to be applied;
- Cure the resin using 20 to 30 seconds of light from a curing light. It is recommended to proceed with polymerization immediately after applying the resin to prevent any possible irritation;
- Check that the product covers the entire surface of the gum and that it has completely cured. To observe the posterior part of papillae and gingiva use a clinical mirror looking from incisal to cervical. Avoid testing it by forcing or trying to lift it, as this could cause infiltration of the irritating substance;
- To remove the product, simply detach it from the front. It should easily come off completely or in large chunks.

8. CONTRAINDICATIONS

The product has methacrylate monomers that may cause allergic reactions or mild irritations to patients sensitive to these substances. Discontinue use of the product in case of allergic reactions

9. WARNINGS

Incomplete polymerization of the product or excessive time may lead to mild gingival irritation in some cases. Side effects

10. SPECIAL CARES

Avoid applying the product on the gums of patients who have recently undergone some type of oral surgical procedure or who have any type of gingival irritation.

11. MANIPULATION

Aseptic environment.

12. INTERACTIONS

There is not.