

## INSTRUCTIONS FOR USE

# PROPHYLACTIC PASTE

### 1. PRODUCT DESCRIPTION

Prophylactic Paste AF do BRASIL has refreshing flavors and contains fluoride, appropriately dosed abrasives, promoting greater protection against cavities in prophylaxis procedures and quality polishing.

### 2. PRESENTATION

- Tube of Prophylactic Paste 90 gr in tutti-frutti flavor;
- Tube of Prophylactic Paste 90 gr in mint flavor;
- Tube of Prophylactic Paste 90 gr in spearmint flavor.

### 3. COMPOSITION

Water, sodium fluoride, carboxymethylcellulose, sorbitol, sodium lauryl sulfate, silica, calcium carbonate, quartz, methyl paraben, sodium saccharin, vegetable glycerin and aroma.

### 4. INDICATION FOR USE

It is indicated for prophylaxis, cleaning procedures, removing debris, stains, bacterial plaque and polishing, as part of the professional prophylaxis treatment for cavities and periodontal diseases.

### 5. CONSERVATION/STORAGE

Keep the product in its packaging always closed;

Protect from direct sunlight;

Store the Product in a cool, dry place at a temperature of 5°C to 30°C.

### 6. INSTRUCTIONS FOR USE

1. Remove from the tube and place the necessary amount of prophylactic paste in a clean AF do BRASIL dappen jar. Close the tube immediately after removing the material, in order to avoid changing the consistency of the product.
2. With a low speed motor, drive the rubber cup (or robinson brush) AF do BRASIL against the surface of the paste until it is filled.
3. Start prophylaxis buccally and lingually, avoiding excessive pressure and keeping the rubber cup at a 90° angle in relation to the tooth surface.
4. For occlusal purposes, it is recommended to use a Robinson brush for deep grooves, taking care not to injure the gingival margins. In places that are difficult to access, such as on the proximal surfaces, it is recommended to use dental floss or tape impregnated with small amounts of paste.
5. Notes: During cleaning and polishing, the cup (or Robinson brush) must always contain the paste, to avoid excessive heating of the tooth. Control of salivation must be carried out to avoid diluting the paste and swallowing it.

### 7. CONTRAINDICATIONS

There are no contraindications, however, in patients who have a history of allergy to any of the components in the composition, it is recommended not to use it.

### 8. WARNINGS

Product for use by Dental Professionals;

Avoid ingesting the product as it may cause occasional nausea if you drink a lot of water.

### 9. SPECIAL CARES

Do not use if the packaging is damaged/open or out of date.

### 10. VALIDITY

Prophylactic Paste AF do BRASIL is valid for 2 years after the date of manufacture.