

# **INSTRUCTIONS FOR USE**

# VIPLAK Disclosing Agent

## 1. PRODUCT DESCRIPTION

Viplak AF do BRASIL is used to show the presence of bacterial plaques with a reddish color. Composed of basic fuchsin, it has a greater power of staining and revealing bacterial plaque on teeth, even on apparently clean teeth. Due to its revealing action, Viplak favors oral hygiene instruction.

# 2. PRESENTATION

1 vial with I0ml of VIPLAK.

## 3. COMPOSITION

Basic Fuchsin; Ethyl Alcohol and Deionized Water.

## 4. INDICATION OF USE

Indicated to show bacterial plaque.

#### 5. PRODUCT ACTION

VIPLAK gives the product greater power to stain bacterial plaque, on apparently clean teeth, in a color that contrasts with the teeth. This artifice not only facilitates the location of plaque, it also favors oral hygiene instruction.

# 6. CONSERVATION/STORAGE

STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

# 7. INSTRUCTIONS FOR USE IN ENDODONTICS

- 1. Topical application:
- Moisten the swab or cotton ball with VIPLAK and apply to all dental surfaces;
- Then make a mouthwash with water to remove the excess.
- 2. Mouthwash:
- Place approximately 6ml of water in a glass, add 1ml (approximately 16 drops) of VIPLAK, rinse for 1 minute and dispense; Then
  rinse your mouth with water to remove excesses.

NOTE:

- All areas that become stained identify the presence of bacterial plaque, which must be removed by using a toothbrush and dental floss.
- VIPLAK can be used daily, preferably during the last tooth brushing.

#### 8. CONTRAINDICATIONS

Contraindicated for people with reported sensitivity to any of the product's components.

## 9. WARNINGS

- Do not use expired expiration date;
- Keep out of the reach of children;
- Exclusive use of the dentist.

# **10. SPECIAL CARES**

Avoid excessive inhalation of concentrated vapors. The use of protective glasses and gloves is recommended.

#### **11. MANIPULATION**

- When handling the product, the professional must wear protective gloves and goggles;
- Dispose of the product in accordance with current legislation in the country.

# **12. INTERACTIONS**

There is not.

- Rua Damaris Rosa de Menezes Monteiro, 136
   Da Tata Francisco Caisara Unicadái 100021 236 Una adicidade de Caisara Una adicidade de Cais
- Pq. Tec. Francisco Sciarra | Lindóia | 86031-216 | Londrina/PR



🛛 🖓 @afdobrasil 🛛 🛛 @afdobrasil