

INSTRUCTIONS FOR USE

CARIOSTATIC

Caries Inhibiting Agent

1. PRODUCT DESCRIPTION

Cariostatic AF do BRASIL is a 12% or 30% solution of silver diamine fluoride. It is widely used in dentistry as an inhibitor of dental caries, this preventive protection is caused by the presence of silver. Also having a preventive remineralization action, as a curative action, in the case of paralysis of carious lesions and antimicrobial action.

2. PRESENTATION

1 vial with 5 or 10ml of CARIOSTATIC 12% and 30%.

3. COMPOSITION

Silver Nitrate, Ammonium Hydroxide, Hydrofluoric Acid and Aqueous Vehicle.

4. INDICATION OF USE

Indicated as an inhibitor of dental caries, preventing the formation of bacterial plaque and the demineralization of the enamel surface. Prevents the progression of caries already installed. Used especially for caries in children and night bottle. It acts on both enamel and dentin.

5. PRODUCT ACTION

The treatment of caries with silver diamine fluoride solution, which is cariostatic and preventive, not being invasive and therefore without cavity restoration, is an important ally in the recovery of oral health in younger children. Silver diamine fluoride forms stable compounds of calcium fluoride, silver phosphate and silver proteinate with the tooth surface, which provide greater resistance to the tooth structure.

6. CONSERVATION/ STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

7. INSTRUCTIONS FOR USE IN ENDODONTICS

- Perform tooth cleaning;
- Protect surrounding soft tissues with Vaseline or absolute isolation;
- Apply the product (2-3 minutes);
- Brush the tooth.

Form of treatments:

- Initial treatment: carry out one application per week for four weeks;
- Maintenance treatment: carry out an application every six or twelve months, depending on the need.

8. CONTRAINDICATIONS

There are no contraindications.

It should not be used in people with proven sensitivity to the product.

9. WARNINGS

- Do not ingest;
- Do not use after expiration date;
- Keep out of the reach of children;
- For dental use only;
- Handle according to the instructions.

10. SPECIAL CARES

- Prevent contact with skin and eyes (caustic product);
- Prevent contact with soft tissue. Isolate with rubber dam or Vaseline;
- The product causes dark spots. Protect hands, clothing and other contact areas during product application and do not handle the packaging;
- Use iodine solution to remove stains caused by cariostatic on the skin.

11. MANIPULATION

After use, keep the package tightly closed.

12. INTERACTIONS:

There is not.