

INSTRUCTIONS FOR USE

PRO TOOTH WHITENER

35% Hydrogen Peroxide - KIT 1 Patient

1. PRODUCT DESCRIPTION

35% Hydrogen Peroxide Pro Tooth Whitening - KIT 1 Patient AF do BRASIL is a Professional dental whitening system based on 35% Hydrogen Peroxide for exclusive use in Dental Offices. A product that has a shorter treatment time due to its single application. After homogeneous mixing of the two phases, a gel based on 35% Hydrogen Peroxide is formed, with pH in the range 6.0 to 7.0. Its neutral pH prevents and reduces effects on the tooth structure, in addition, Pro Tooth Whitening AF do BRASIL contains desensitizing agents in its formula that maximize patient comfort. It should always be used with an efficient isolation system for gingival protection (Photopolymerizable Gingival Barrier AF do BRASIL)

2. PRESENTATION

Box with 1 (one) 35% Hydrogen Peroxide Pro Tooth Whitening - KIT 1 Patient:

- 1 (one) hydrogen peroxide syringe containing 1.2g;
- 1 (one) 0.4g thickener syringe;
- 1 (one) 3.0g photopolymerizable gingival barrier syringe;
- 1 (one) black applicator tip;
- 1 (one) transparent applicator tip;
- 1 (one) intermediate connector.

3. COMPOSITION

Hydrogen peroxide, Stabilizer, Neutralizing Agent, Thickener, Desensitizing Agents, Structuring, Preservatives, Blue Pigment, Enzyme and Deionized Water.

4. INDICATION OF USE

Pro Tooth Whitener is for use in the office and indicated for vital and non-vital teeth with extrinsic color changes of a chronic nature, darkened, not amenable to removal by conventional procedures for exclusive use in the dental office. It can be used with blue light energy sources to assist in accelerating the reaction of the tooth whitening process.

5. PRODUCT ACTION

Pro Tooth Whitener has the action of the whitening agents penetrating the dental structures and their reaction with the pigmented molecules, oxidizing and fractionating into smaller and smaller molecular chains, causing the pigments incorporated to the tissues to be totally or partially eliminated of the tooth structure by diffusion.

6. CONSERVATION/ STORAGE

- Keep the product in its packaging always closed;
- Protect from direct sunlight;




STORE THE PRODUCT AT TEMPERATURES BETWEEN 5°C AND 28°C. AND DO NOT FREEZE THE PRODUCT OR LEAVE IT EXPOSED TO HIGH TEMPERATURES.

7. INSTRUCTIONS FOR USE

- Clean the teeth, after that record the initial color using the Vita scale or photography;
- Dry the teeth well and perform a careful relative isolation with Photopolymerizable Gingival Barrier AF do BRASIL, contouring the gingival tissue and light curing each tooth to be whitened for 10 seconds;
- Connect the two syringes of the Self-Mixing Pro Tooth Whitening Kit, after complete and safe coupling, mix by pushing the plungers alternately for up to 10 times or until a homogeneous mixture is presented, below is an explanation of the self-mixing system:

AUTOMIXING SYSTEM

The self-mixing system is used to facilitate mixing the peroxide with the thickener, using two syringes connected by a system that allows the gel to pass from one syringe to the other, effectively mixing the two phases. After mixing, put the mixed gel in just one syringe and use it. Figure 01 presents a schematic illustration of the operation of self-mixing.

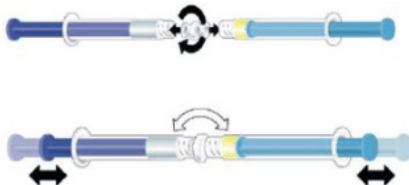

- Start applying the whitening gel already mixed on the entire buccal and proximal surface of each tooth to be whitened. The gel layer should be 1mm thick;
- The mixed whitening gel should remain on the teeth for up to 45 minutes. If you need to release any oxygen bubbles, use a disposable brush. If the patient reports sensitivity, remove the product and assess the reason (crack, infiltration, exposed dentin, etc.);
- After 45 minutes, remove the whitening gel by aspirating with an appropriate suction tip and wash thoroughly with water;
- Remove the Photopolymerizable Gingival Barrier AF do BRASIL by detaching with an explorer probe. Apply Dental Desensitizer AF do BRASIL for 10 minutes.

8. CONTRAINDICATIONS

- Promote a detailed anamnesis of the patient before starting treatment;
- Handle the product only in a dental office, using protective gloves and goggles;
- Avoid contact of the gel with soft tissues and gums;
- The product is not recommended for people who have recently had gum surgery or with inflammation of the gum tissue and people with allergies to any of its components.
- It is not recommended to use during pregnancy, lactation and under 15 years.
- Product for exclusive use in a dental office under the supervision of a professional;
- Non-sterile product;
- Read the instructions for use before using the product;
- Do not use after expiration date;
- Do not ingest and avoid contact with skin, oral soft tissues, and eyes;
- It is recommended that the patient wear protective eyewear.

9. SIDE EFFECTS

- Ingestion of large amounts of the product may cause nausea;
- Before starting treatment, inform your patient about the possibility of tooth sensitivity. Dental sensitivity is usually common during whitening treatment. In this case, if the sensitivity persists, treat the teeth with AF do BRASIL Dental Desensitizing Gel 2%, which is indicated for moderate to high sensitivity;
- Suspend the use of the product in case of allergic reactions.

10. MANIPULATION

- When handling the product, the professional must wear protective gloves and goggles;
- Dispose of the product in accordance with current legislation in the country.