

## INSTRUCTIONS FOR USE

# DENTAL WEDGES

## Wooden Dental Wedges

### 1. PRODUCT DESCRIPTION

Dental Wedges AF do BRASIL are made of wood and colored with non-toxic pigments. They guarantee perfect fixation of the matrices and do not require cuts or adjustments, resulting in restorations free of subgingival excess and with perfect contour. Symmetrical and anatomical geometry. Easy handling with tweezers and does not stain the restoration.

### 2. PRESENTATION

Wooden wedges containing 100 colored units.

### 3. COMPOSITION

Natural wood and colored by non-toxic and non-flammable reactive dyes.

### 4. CONSERVATION I STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE(15 TO 30C).

### 5. INSTRUCTIONS FOR USE

- Select the desired wedge.
- Insert the wedge into the interproximal space.
- Once the procedure is completed, the wedge must be removed.

### 6. WARNINGS

- Professional use.
- Single use.
- Its reuse can cause cross-contamination.
- It is recommended that the use of the product always be accompanied by absolute isolation of the operative field, since
- Loose objects can be accidentally swallowed or aspirated.
- The product has a pointed end and must be handled in such a way as not to cause injury to the patient.
- When there is proven allergy to any of its components, the product should not be used.
- Any serious device-related incident should be reported to the manufacturer.

### 7. INTERACTIONS

There is not.

