

## INSTRUCTIONS FOR USE

# ROBINSON'S BRUSH

### 1. PRODUCT DESCRIPTION

- The Robinson Brush is indicated for prophylaxis, polishing and cleaning procedures of the dental arch.
- Soft and Abrasive Bristles made of 6.0 nylon Monofilament measuring 018mm;
- In White, Colored or Black colors;
- Rod for Contra Angle (CA).

### 2. PRESENTATION

- CONICAL ROBINSON BRUSH CA WHITE presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;
- CONICAL ROBINSON BRUSH CA COLORED presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;
- CONICAL ROBINSON BRUSH CA BLACK presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;
- FLAT ROBINSON BRUSH CA WHITE presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;
- FLAT ROBINSON BRUSH CA COLORED presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;
- FLAT ROBINSON BRUSH CA BLACK presentation can be with 1 unit, 5 units, 10 units, 50 units or 100 units;

### 3. COMPOSITION

- Brass – CLA 360, Standard STM 316;
- Chromium Nickel;
- Nylon 6.0 monofilament measuring 018 mm.

### 4. INDICATION OF USE

Indicated for prophylaxis, polishing and cleaning of the dental arch.

### 5. CONSERVATION/STORAGE

- Keep the product in its packaging always closed;
- Humidity must be maintained in a range between 35% and 70% RH.



STORE THE PRODUCT IN A COOL DRY PLACE AT TEMPERATURE (15°C TO 30°C).

### 6. INSTRUCTIONS FOR USE

Choose your preferred format for the tooth surface in question. Connect the rod (mandrel) of the product to the low-speed motor. To carry out the prophylaxis procedure, use a prophylactic paste or abrasive powder as an aid.

### 7. CONTRAINDICATIONS:

There is not.

### 8. WARNINGS

- Do not ingest;
- Keep out of the reach of children;
- For Dental use only;
- Handle according to the instructions;
- Sterilize before use.

### 9. SPECIAL CARES

Professional use, single use, non-sterile, discard after use.

### 10. MANIPULATION

Aseptic environment.

### 11. INTERACTIONS

There is not.