

INSTRUCTIONS FOR USE

EUCALYPTOL

Gutta Percha Solvent

1. PRODUCT DESCRIPTION

Eucalyptol AF do BRASIL is obtained from eucalyptus essential oil. It is a clear, colorless liquid, with an aromatic, camphoraceous, characteristic odor and a spicy taste, followed by a feeling of freshness. Practically insoluble in water, miscible with alcohol, chloroform, carbon sulfide, glacial acetic acid, vegetable oils and essences. It is a liquid that helps in the retreatment of root canals, acting as a solvent for gutta percha cones.

2. PRESENTATION

1 vial with 10ml of EUCALYPTOL.

3. COMPOSITION

EUCALYPTOL 100%.

4. INDICATION OF USE

Indicated as a solvent for gutta percha cones, aiding in the removal of fillings from root canals.

5. PRODUCT ACTION

Gutta Percha are obturator cones for root canals, intended to occupy the space created by the instruments.

Gutta percha is a white amorphous substance, flexible, not malleable at temperatures below 50°C and soluble in eucalyptol.

When solubilized in eucalyptol, gutta percha becomes pasty and is used to fill the canal. These pastes harden with the volatilization of the solvents.

Fillings made with gutta percha cones are easily removed by using eucalyptol, xylene or chloroform, which dissolve them with the aid of the action of instruments.

6. CONSERVATION/ STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE
(15 TO 30°C).

7. INSTRUCTIONS FOR USE

- Dispense a drop of Eucalyptol into the root canal;
- Wait around 5 minutes for the gutta percha cone to soften;
- With the aid of an endodontic file instrument and remove the gutta percha;
- Dispense another drop of Eucalyptol when you notice that the remaining gutta percha cone is dry or brittle;
- Repeat the operation until complete removal.

8. CONTRAINDICATIONS

- There are no contraindications;
- It should not be used in people with proven sensitivity to the product.

9. WARNINGS

- Avoid contact with skin and eyes;
- Do not ingest;
- Do not use after expiration date;
- Keep out of the reach of children;
- For dental use only;
- Handle according to the instructions.

10. SPECIAL CARES

- Avoid contact with skin and eyes;
- The use of protective goggles and gloves is recommended.

11. INTERACTIONS

There is not.