

INSTRUCTIONS FOR USE

DISPOSABLE DOUBLE TRAYS

1. PRODUCT DESCRIPTION

Disposable Double Trays AF do BRASIL are made of white polystyrene and excellent quality polyurethane foam. They have a universal fitting, precision in application and handling.

2. PRESENTATION

Package containing 24, 50 or 100 units;

Available in the following models:

- Disposable Double Tray Size S;
- Disposable Double Tray Size M;
- Disposable Double Tray Size G;
- Disposable Double Tray Size MIXED.

3. COMPOSITION

White Polystyrene and Polyurethane Foam.

4. INDICATION FOR USE

Indicated for fluoride application

5. STORAGE

Store in a cool, clean and dry place, away from direct sunlight.

6. INSTRUCTIONS FOR USE

- Dry the dental surfaces;
- Place the fluoride gel on the side with the tray foam;
- Introduce the Disposable Double Trays AF do BRASIL in the patient's mouth, so that the fluoride foam is in contact with the teeth;
- Ask the patient to gently bite the tray to better fit the teeth;
- Leave the fluoride in contact with the teeth according to the instructions for use of fluoride gel;
- Remove the tray and remove excess gel.

7. CONTRA INDICATIONS

Do not use in patients with any sensitivity to the components.

8. WARNINGS

Always check if the product is in perfect condition and free of defects;

9. SPECIAL CARE

Not applicable.

10. VALIDITY

The Disposable Double Trays AF do BRASIL are valid for 05 years, after the manufacturing date.