

## INSTRUCTIONS FOR USE

# ORANGE OIL

### 1. PRODUCT DESCRIPTION

Orange Oil AF do BRASIL is used in endodontic treatment, helping to clear root canals.

### 2. PRESENTATION

1 bottle with 10ml of Orange oil.

### 3. COMPOSITION

Orange Oil (Terpenode Orange).

### 4. INDICATION OF USE

Indicated for endodontic treatment as a solvent for Gutta-Percha and obturator cements, helping to clear root canals.

### 5. PRODUCT ACTION

Orange oil is a chemical solvent extracted from the peel of the sweet orange, it is volatile and its main characteristic is that it is not toxic to professionals, patients and the environment.

In dentistry, it is used as a solvent for gutta-percha and obturator cements, in addition to being effective in clearing root canals filled with eugenol zinc oxide cement.

### 6. CONSERVATION/ STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

### 7. INSTRUCTIONS FOR USE IN ENDODONTICS:

- Remove the restoration by mechanical method up to the entry of the root canal; Dispense 1 to 2 drops of Orange Oil at the root canal entrance and wait for 5 to 10 minutes;
- After the action time, with the aid of an endodontic file or reamers, perform light penetration movements and introduce the instrument inside the cement, performing the removal;
- Dispense the Orange Oil as necessary;
- At the end of removing the obturator material, irrigate the canal with surfactant solution.

### 8. CONTRAINDICATIONS

Contraindicated for people with reported sensitivity to any of the product's components.

### 9. WARNINGS

- Do not use after the expiry date;
- Keep out of the reach of children;
- Exclusive use of the dentist.

### 10. SPECIAL CARES

- Avoid excessive inhalation of concentrated vapors;
- The use of protective goggles and gloves is recommended.

### 11. MANIPULATION

- When handling the product, the professional must wear protective gloves and goggles;
- Dispose of the product in accordance with current legislation in the country.

### 12. INTERACTIONS

There is not.