

## INSTRUCTIONS FOR USE

# CARBON PAPER

## Carbon Paper for Joints

### 1. PRODUCT DESCRIPTION

Carbon Paper AF do BRASIL double-sided strip-shaped in blue and red colors. Covered with a thin layer of paraffin, it has resistance to moisture and traction.

### 2. PRESENTATION

Block of 12 sheets with carbon paper blades for articulation in the form of a strip or a horseshoe for full arches, double-sided in blue/blue, blue/red.

### 3. COMPOSITION

Special papers; Vegetable Waxes; Pigments; Vegetable Oils; Paraffin and Vaseline.

### 4. INDICATION OF USE

Indicated for registering occlusal contacts in adjustment procedures for restorations, prosthetic parts and dental surfaces.

### 5. PRODUCT ACTION

CARBON PAPER is used to mark occlusal points, allowing better identification of regions for finishing restorations and dental surfaces.

### 6. CONSERVATION/STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

### 7. INSTRUCTIONS FOR USE

- Insert the CARBON PAPER into the patient's mouth or into the plaster model;
- Ask the patient to occlude to mark the occlusal contact points;
- Make adjustments with suitable instruments if necessary.

**IMPORTANT:** To obtain a better result from the product, dry the dental surfaces to allow the carbon paper ink to mark the contact points clearly and without smudging.

### 8. CONTRAINDICATIONS

The product has no contraindications.

### 9. WARNINGS

- Keep out of the reach of children;
- For dental use only;
- Handle according to the instructions;
- Apply the product using gloves, mask and goggles;
- Do not ingest;
- Use only once.

### 10. SPECIAL CARES

There is not.

### 11. MANIPULATION

After use, keep the product block protected from light, heat and humidity.

### 12. INTERACTIONS

There is not.