

INSTRUCTIONS FOR USE

PUMICE STONE

Extra Fine Pumice Stone

1. PRODUCT DESCRIPTION

Pumice Stone AF do BRASIL is an extra fine abrasive powder, used for cleaning and polishing teeth and metal restorations and prostheses. The product is used as an abrasive and employed in cleaning and polishing dental enamel, cohesive gold, amalgam and acrylic resin.

2. PRESENTATION

1 bottle with 100g of pumice stone.

3. COMPOSITION

PUMICE STONE.

4. INDICATION OF USE

Indicated for cleaning and polishing dental enamel, cohesive gold, amalgam and acrylic resin.

5. PRODUCT ACTION

Pumice Stone is an abrasive material, which promotes the removal of external bacterial plaques and surface stains.

6. CONSERVATION/STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

7. INSTRUCTIONS FOR USE

Apply the product to the tooth, pure or with water, using rubber cups.

8. CONTRAINDICATIONS

There are no contraindications.

9. WARNINGS

- Do not ingest;
- Do not use if the seal is broken;
- Do not use after expiration date;
- Keep out of the reach of children; For dental use only;
- Handle according to the instructions;
- Apply the product using gloves, mask and goggles;
- Avoid inhalation and contact with skin and eyes.

10. SPECIAL CARES

Avoid inhalation and contact with skin and eyes. It is recommended to use gloves, mask and goggles.

11. MANIPULATION

After use, keep the package tightly closed.

12. INTERACTIONS

There is not.

