

## INSTRUCTIONS FOR USE

# RETRAF

## Gingival Retractor Wire

### 1. PRODUCT DESCRIPTION

The Retraf AF do BRASIL is a gingival retractor wire made of 100% braided cotton thread. Due to its composition it has a good absorption, they also have a great elastic pressure on the gingival margin, adequately moving away the tissue in the sulcus and helping the professional in his procedures.

### 2. PRESENTATION

Bottle containing 250 cm of RETRAF.

They have 5 Models:

- 000 - Ultra Extra Slim - Black.
- 00 - Extra Fine - Yellow.
- 0 - Thin - Lilac.
- 1 - Medium - Blue.
- two - Thick - Green.
- 3 - Extra Thick - Red.

### 3. COMPOSITION

100% cotton yarn braided.

### 4. INDICATION OF USE

Indicated to retract the gingiva in several dental procedures, it facilitates the handling of dental infusions, access and visibility of the entire gingival tissue, in addition to helping to prevent damage to the gingival tissue during the preparation of cavity preparations.

### 5. PRODUCT ACTION

The RETRAF gingival retractor wire produces a mechanical action, making a gingival/tooth surface interposition. With normal tissue moisture, water and/or gingival recession solution, the braided retractor wire promotes gingival tissue retraction.

### 6. INSTRUCTIONS FOR USE

- Cut a piece of wire to the desired length;

Note: The wire may or may not be soaked in a hemostatic/retractor solution to obtain greater retraction;

- Position the wire around the neck of the tooth, and introduce it into the groove, carefully compressing it using a spatula without a sharp edge. (Start at the interproximal space where the sulcus is usually deepest and then proceed around the tooth completely in a clockwise direction);
- If necessary, to facilitate removal, slightly moisten the retractor cord with water.

Note: Do not remove the wire immediately before taking the impression.

### 7. IDEAL TIME FOR PERMANENCE OF THE PRODUCT

- If the hemostatic/retractor solution is not used, the ideal time for the product to remain in order to obtain retraction is at least 5 minutes;
- When using hemostatic/retractor solution, follow the time indicated in the solution manufacturer's instructions for use;
- In moldings, leave the wire positioned until immediately before taking the impression.

### 8. WARNINGS

- Do not ingest;
- Do not use after expiration date;
- Keep out of the reach of children;
- For dental use only;
- Use the product only on healthy gums;
- Use the product as indicated in the Instructions for Use;
- Product for professional use only.

### 9. PRECAUTIONS

- Use on healthy tissues;
- The impregnation of hemostatic solution must be taken into account the instructions of the manufacturer of the solution used;
- After using the RETRAF gingival retractor wire, check and inspect carefully to verify that there are no fragments of the gingival retractor wire;
- Discard after use.

### 10. STORAGE AND TRANSPORT CONDITIONS



STORE IN A COOL AND DRY PLACE AT ROOM TEMPERATURE (15° TO 30°C).

- Humidity must be maintained in a range between 35% RH and 70% RH;
- It does not require special transport conditions. Transport in the original packaging of the product.

### 11. CONTRAINDICATIONS

It is contraindicated for people who are sensitive to cotton.