

INSTRUCTIONS FOR USE

RUBBER CUP

Rubber Cup for Prophylaxis and Polishing

1. PRODUCT DESCRIPTION

The Rubber Cup is indicated for prophylaxis, polishing and cleaning of the dental arch.

- In White or Colored colors;
- Rod for Contra Angle (CA).

2. PRESENTATION

- RUBBER AND SILICONE CUP FOR PROPHYLAXIS AND POLISHING in WHITE color presentation can be with 1 unit, 7 units, 8 units, 50 units or 100 units;
- RUBBER AND SILICONE CUP FOR PROPHYLAXIS AND POLISHING COLORED presentation can be with 1 unit, 7 units, 8 units, 50 units or 100 units.

3. COMPOSITION

- Brass - CLA 360, Standard STM 316;
- Chromium Nickel;
- BDP rubber;
- Silicone.

4. INDICATION OF USE

Indicated for prophylaxis, polishing and cleaning of the dental arch.

5. PRODUCT ACTION

It presents a high performance in polishing and cleaning the dental arch.

6. CONSERVATION/STORAGE

- Keep the product in its packaging always closed;
- Humidity must be maintained in a range between 35° % and 70° RH.



STORE THE PRODUCT IN A COOL DRY PLACE AT TEMPERATURE (15°C TO 30°C).

7. INSTRUCTIONS FOR USE

It is placed in the high-speed pen and then the pen is activated to carry out the prophylaxis procedure, using as an auxiliary prophylactic paste or abrasive powder.

8. CONTRAINDICATIONS

There is not.

9. WARNINGS

- Do not ingest;
- Keep out of the reach of children;
- For Dental use only;
- Handle according to the instructions;
- Sterilize before use.

10. SPECIAL CARES

Professional use, single use, non-sterile, discard after use.

11. MANIPULATION

Aseptic environment.

12. INTERACTIONS

There is not.