

INSTRUCTIONS FOR USE

POLYESTER MATRIX STRIPS

1. PRODUCT DESCRIPTION

The Polyester Matrix Strip AF do BRASIL is made of 100% high quality and super flexible polyester.

2. PRESENTATION

- Envelope Containing Polyester Matrix Strips in model:
- Polyester Matrix Strip 10mmx120mmx0.05mm containing 50 units.

3. COMPOSITION

- Non-flammable and non-toxic polyester.

4. INDICATION OF USE

Indicated to be positioned between the teeth to promote a separation between them, thus preventing the restorative material from being deposited on the adjacent tooth, and can also be used as a shaper to outline the restorations.

5. PRODUCT ACTION

The purpose of the Polyester Matrix Strips is to act as a physical barrier to the restorative material.

6. CONSERVATION / STORAGE:



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 - 30°C).

7. INSTRUCTIONS FOR USE

- Cut the ribbon to the desired size;
- Introduce the tape in the matrix holder;
- Insert and adjust the set to the tooth to be restored;
- Adjust the matrix to the tooth;
- Insert the anatomical wedge into the embrasure and perform the restorative procedure;
- After completing the restoration, remove the wedge and tape.

8. CONTRAINDICATIONS

AT

9. WARNINGS

- Do not ingest;
- Keep out of the reach of children;
- For professional use only;
- Handle according to the instructions;
- single use
- When there is an allergy to the component, suspend its use.

10. SPECIAL CARES

Keep the product in the original packaging until use.

11. MANIPULATION

When handling the product, the professional must wear gloves, a mask and goggles.

12. INTERACTIONS

There is not.