

INSTRUCTIONS FOR USE

STAINLESS STEEL ABRASIVE STRIPS

1. PRODUCT DESCRIPTION

Stainless Steel Abrasive Strips AF do BRASIL are produced in stainless steel and developed for professional dental use. They do not break and are very flexible. They have a spectacular differential that is their NEUTRAL CENTER, without abrasive, facilitating the introduction on the Interproximal surfaces of the teeth, bringing greater precision and comfort to the patient.

2. PRESENTATION

Box containing 12 units or Package containing 100 units;

Available in the following models:

- Abrasive Steel Strips 4mm x 140mm;
- Abrasive Steel Strips 6mm x 140mm;

3. COMPOSITION

Stainless Steel and Aluminum Oxide.

4. INDICATION FOR USE

It is indicated for finishing and polishing proximal surfaces of restorations made with composites, acrylics, glass cement ionomers and amalgams.

5. STORAGE

Store the product in a cool, clean and dry place, away from direct sunlight.

6. INSTRUCTIONS FOR USE

- Carefully take the strips to the proximal surfaces of the restoration;
- Introduce them into the interproximal spaces of the tooth through the Neutral Center;
- After starting the grinding in the treated area.

7. CONTRA INDICATIONS

Do not use in patients with any sensitivity to the components.

8. WARNINGS

Always check that the product is in perfect condition and contains no defects;

9. 9. SPECIAL CARE

Not applicable.

10. 10. VALIDITY

Stainless Steel Abrasive Strips AF do BRASIL are valid for 5 years after the manufacturing date.