

INSTRUCTIONS FOR USE

ABRASIVE POLYESTER STRIPS

1. DESCRIPTION OF THE PRODUCT

Abrasive Polyester Strips AF do BRASIL are produced in 100% polyester and aluminum oxide abrasives, developed and designed for professional dental use, with two (2) granulations, medium (white) and coarse (gray). The strips have a Neutral Center, without abrasive, facilitating the introduction in the Interproximal surfaces of the teeth.

2. PRESENTATION

Box containing 50 or 150 units;

Available in the following models:

- Abrasive Polyester Strips 2mm x 170mm;
- Abrasive Polyester Strips 4mm x 170mm.

3. COMPOSITION

Polyester and Aluminum Oxide.

4. INDICATION FOR USE

Indicated for finishing the proximal faces of resin restorations.

5. STORAGE

Store in a dry place away from sunlight.

6. INSTRUCTIONS FOR USE

The strip should be held by the pulps of thumb and forefinger, after penetrating the interdental space, by the Neutral Center it should be moved from vestibular to lingual and vice-versa, from the region of the servical joint towards the occlusal surface of the tooth.

7. CONTRA INDICATIONS

Do not use in patients with any sensitivity to the components.

8. WARNINGS

Always check if the product is in perfect condition and free of defects;

9. SPECIAL CARE

Not applicable.

10. VALIDITY

Abrasive Polyester Strips AF do BRASIL are valid for 5 years after the manufacturing date.