

INSTRUCTIONS FOR USE

VIPLAK DUO Blue and Pink Disclosing Agent

1. PRODUCT DESCRIPTION

The VIPLAK DUO Blue and Pink AF do BRASIL is used to show the presence of bacterial plaques, with the aid of two pigmentations to indicate old plaque and new plaque.

2. PRESENTATION

1 vial with 5ml, 10ml, 15ml, 20ml, 30ml, 50ml or 1000ml.

3. COMPOSITION

Brilliant Blue Pigment, Erythrosine Red Pigment, Methylparaben, Propylparaben, Tuti Frutti Aroma and Deionized Water.

4. INDICATION OF USE

Indicated to show bacterial plaque with two chromatic indicators to color old plaque (blue) and new plaque (pink).

5. PRODUCT ACTION

VIPLAK DUO Blue and Pink AF do BRASIL has a greater power of staining bacterial plaque, on apparently clean teeth, having two colors that contrast with the teeth and make it possible to verify old plaques and new plaques. This artifice not only facilitates the location of plaque, it also favors oral hygiene instructions.

6. CONSERVATION/STORAGE



STORE IN A DRY PLACE AT ROOM TEMPERATURE (15 TO 30°C).

7. INSTRUCTIONS FOR USE IN ENDODONTICS

1. Apply 4 to 5 drops in the patient's mouth, after that, ask them to spread it over the teeth with the tongue, and let it act for approximately 10 to 15 seconds.
2. Ask the patient to rinse the mouth until the water is clear of color. All dental surfaces where there is old bacterial plaque will be dyed blue and where there is recent plaque, pink.
3. Carry out the prophylaxis, as well as instruct the patient in the brushing technique, to eliminate plaque until there is no more pigmentation on the blue or pink teeth.
4. Apply VIPLAK DUO Blue and Pink AF do BRASIL again. If prophylaxis or brushing has been perfect, there will be no colored tooth surface.

8. CONTRAINDICATIONS

Contraindicated for people with reported sensitivity to any of the product's components.

9. WARNINGS

- Do not use expired expiration date;
- Keep out of the reach of children;
- Do not ingest, if it occurs, seek a doctor with the leaflet in hand;
- Use PPE (Masks, Goggles and Gloves);
- Exclusive use of the dentist.

10. SPECIAL CARES

Avoid excessive inhalation of concentrated vapors. The use of protective goggles and gloves is recommended.

11. MANIPULATION

- When handling the product, the professional must wear protective gloves and goggles;
- Dispose of the product in accordance with current legislation in the country.

12. INTERACTIONS

There is not.